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Instruction for use OlaCare

 Read the instructions carefully before use, it contains important data.
Keep these instructions for use in a safe place for future reference.

Description:

This is a diaper-rash cream that is designed primarily to prevent skin irritation in places with a risk of sores by acting as a barrier between the skin and the faeces. It contains emulsifier based on the natural proteins extract (protease inhibitors) from *Solanum Tuberosum*, which improves the barrier function of the cream and Panthenol (in a concentration of 1%), which helps to protect the skin through its function as an emollient and humectant.

Intended purpose:

The medical device OlaCare is intended for baby's and infants, for prevention of skin irritation in the diaper area. By applying to the diaper areas, it forms an occlusive lipophilic surface layer on the skin that fulfils a barrier function, reduce skin irritation and friction in the affected areas.

Ingredients:

Aqua, Decyl Oleate, Caprylic / Capric Triglyceride, Glycerin, *Solanum Tuberosum* Callus Culture Extract, Cetearyl Alcohol, Lanolin, Glyceryl Stearate SE, Glyceryl Stearate, Panthenol, Parfum, Sodium Stearoyl Glutamate, Hydrogenated Rapeseed Oil, Hydroxyacetophenone, Polyacrylate Crosspolymer-11, Xanthan Gum, 1,2-Hexanediol, Caprylyl Glycol, Tocopherol, Hydrogenated Vegetable Glycerides Citrate.

Indication:

The cream is intended for daily diaper care to prevent diaper rash. Apply only to areas where there is a risk of diaper rash.

Contraindication:

The cream is not intended for the treatment of injured skin.

Interaction:

Undesirable interactions are not yet known for any component of the composition.

Side effects:

The cream is tested to be hypoallergenic. Yet, in exceptional cases, the cream may cause an allergic skin reaction.

Use and method of application:

After thoroughly cleansing the skin in and around the diaper area, apply a thin layer of cream each time you change the diaper. Pay special attention to the folds of the skin. FOR EXTERNAL USE ONLY.

Packaging:

Tube contains 40 g of cream. The sample sachet contains 5 g of cream.

Notice:

- Do not use if the package has been opened, damaged or heavily soiled.
- Carefully close the tube after each use to prevent contaminating the cream.
- Store in a dry place below 30 °C.
- In case of contact with eyes, rinse thoroughly with water. If irritation continues, seek medical attention.
- Keep out of reach of children.

Warning:

Do not use the cream after the expiry date which is stated on the carton.

 **Manufacturer:** Eurodrug Laboratories B.V., Regulusweg 11,
2516 AC, The Hague, The Netherlands, www.eurodrug.nl
Tel: +31 (0)70416 73 73

Authorized representative in Ukraine:

Medical HUB LLC
16 Sergey Danchenko, office 32, Kyiv, 04078, tel: 38(044) 232 52 52.
Email: uarep@medical-hub.info

Exclusive distributor in Ukraine:

Megacom LTD
195 B Klochkivska, Kharkiv, 61145
Tel: +38(057)-701-37-55, Email: megacom@megacom.com.ua

Edition 3, revision date 10.05.2023